


TRIM-IT™ Custom Hip Cannula

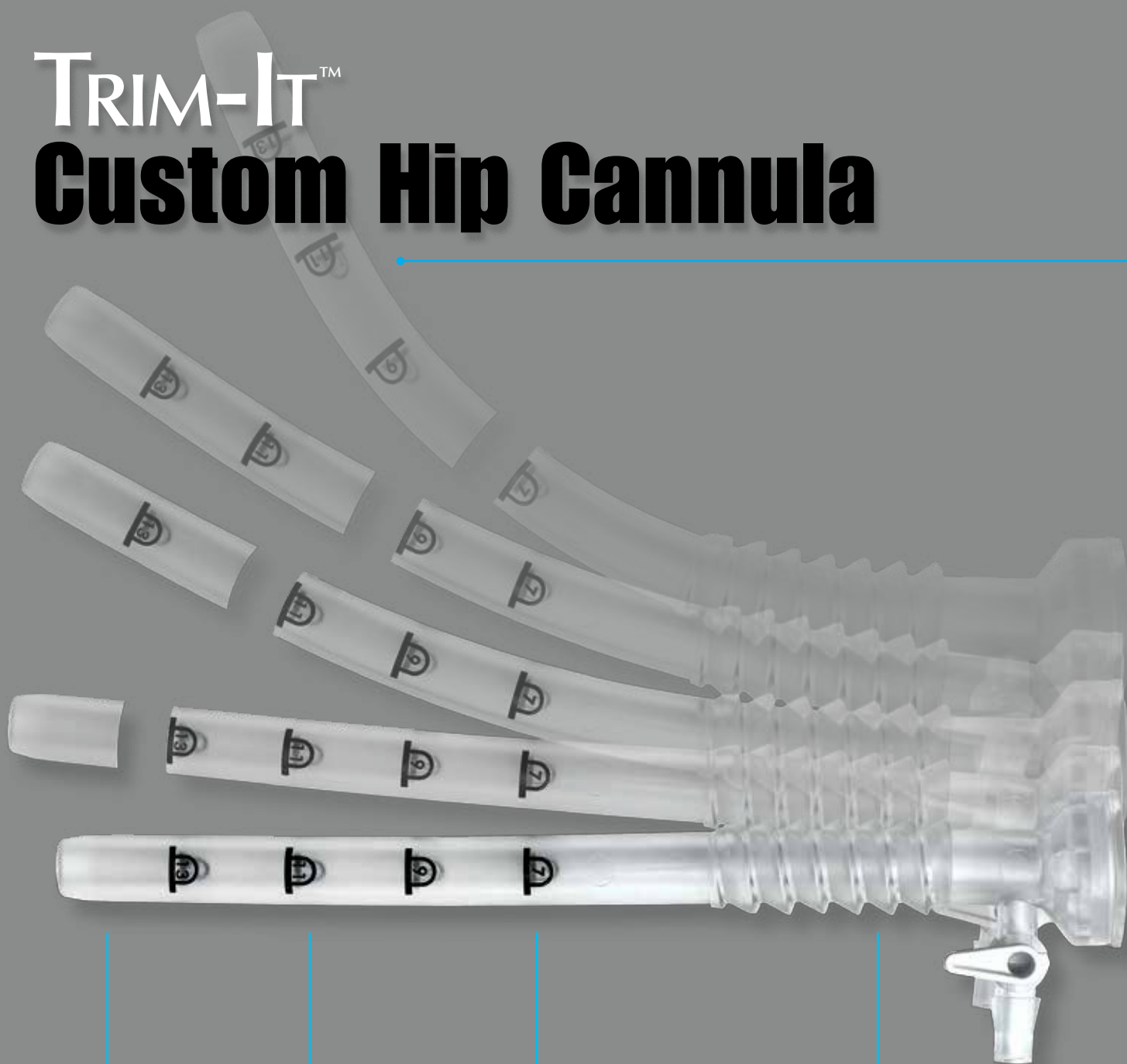

Atraumatic

Flexible

Variable Lengths


Strong Proximal Threads

The 8.25 mm flexible TRIM-IT Custom Hip Cannula is made of a soft polyvinyl chloride plastic designed to provide enhanced instrument mobility and minimize iatrogenic damage to the articular surfaces. The reinforced proximal threads hold exceptionally well in soft tissue and the ability to cut the cannula from 7 cm – 15 cm working lengths minimizes O.R. inventory and makes it one of the most versatile cannulas available.

Simple 3-Step Insertion

1) Measure

Place the calibrated switching stick into the portal and measure the soft tissue distance.

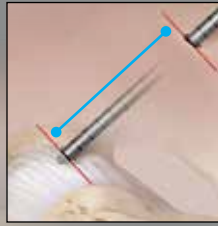


2) Cut

Cut the cannula to the desired length.



3) Insert

Load the cannula onto the adjustable obturator and insert into the joint.

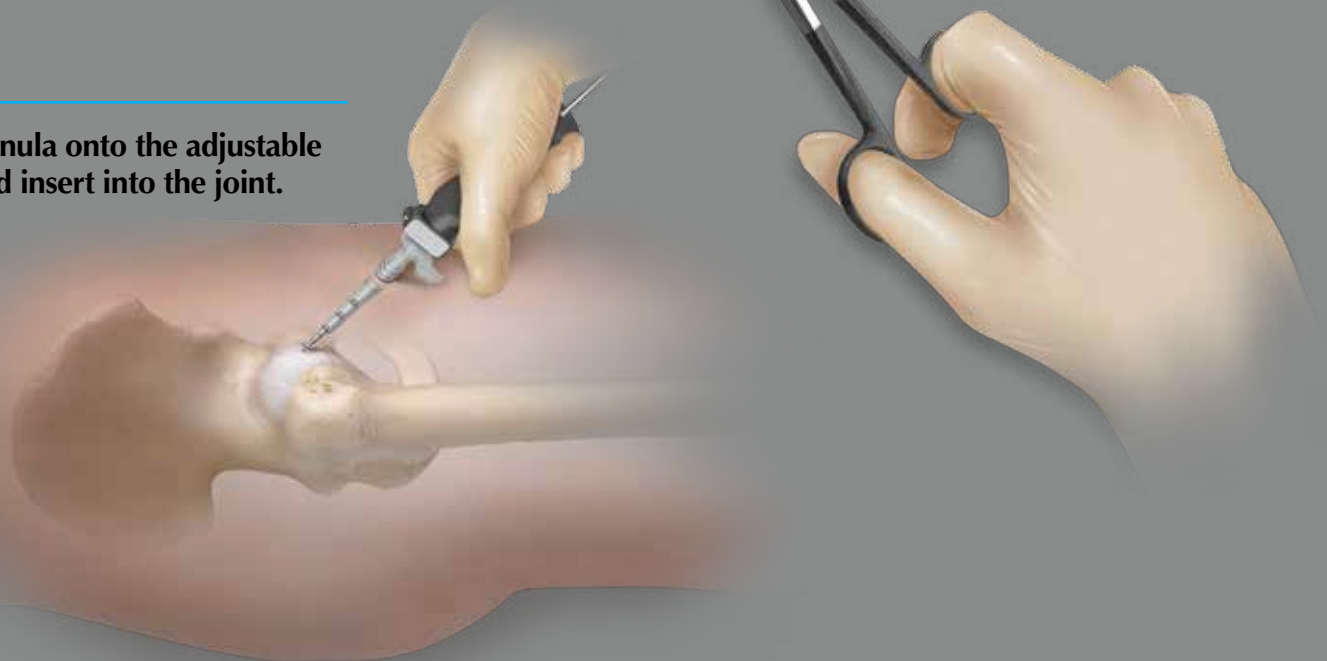

Ordering Information

TRIM-It Custom Hip Cannula, 8.25 mm x 15 cm
TRIM-It Custom Hip Cannula Switching Stick
TRIM-It Custom Hip Cannula Inserter Handle
TRIM-It Custom Hip Cannula Inserter Obturator

AR-6590
AR-6590ST
AR-6590DH
AR-6590DT

Arthrex

www.arthrex.com

©2016, Arthrex Inc. All rights reserved. LS1-00016-EN_A

This description of technique is provided as an educational tool and clinical aid to assist properly licensed medical professionals in the usage of specific Arthrex products. As part of this professional usage, the medical professional must use their professional judgment in making any final determinations in product usage and technique. In doing so, the medical professional should rely on their own training and experience and should conduct a thorough review of pertinent medical literature and the product's Directions For Use.