

AR-6481 Pump Cart

Brochure



AR-6481 Pump Cart

Saline Hanger Assembly Instructions

Hardware Included

- Pole
- Saline Bag Hanger

Installation Instructions

1. Connect the saline bag hanger to the pole by screwing the hanger clockwise into the pole until it is tight
2. Install the provided pole assembly into the left side of the cart by screwing the pole clockwise until tight


DualWave™ and Continuous Wave™ Arthroscopy Pump Assembly to the Top Shelf

Hardware Included

3. 10-32 × 0.75 in Pan Head Machine Screws (12 each)
4. ¼-20 × 0.75 in Pan Head Machine Screws (4 each)

Installation Instructions

- Use the 4 outer holes to mount the DualWave pump with the 10-32 × 0.75 in screws provided
- Use the 4 outer holes to mount the Continuous Wave IV pump with the 10-32 × 0.75 in screws provided
- The 4 inner holes can also be used to mount the Continuous Wave III pump with the ¼-20 × 0.75 in pan head machine screws

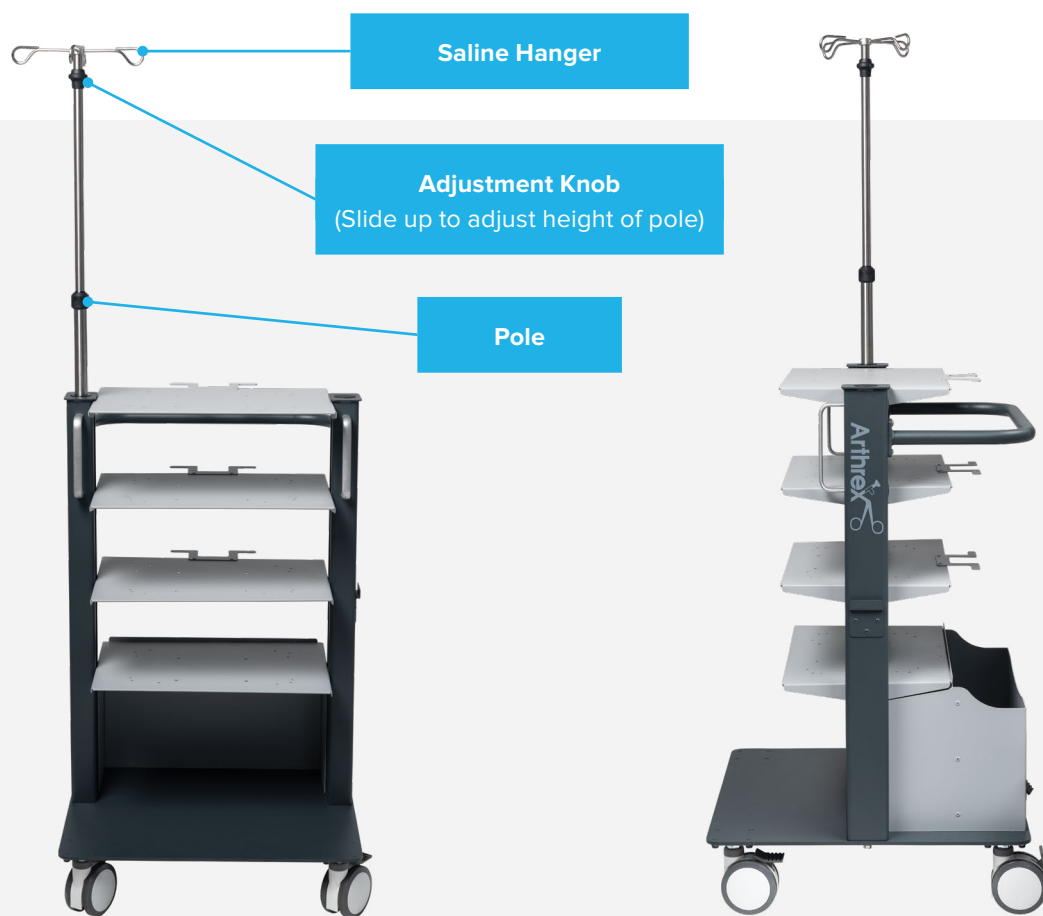

Pole Installation 1

DualWave Pump Mounting Holes 1

Feature Locations



Pump Cart Features 1



This description of technique is provided as an educational tool and clinical aid to assist properly licensed medical professionals in the usage of specific Arthrex products. As part of this professional usage, the medical professional must use their professional judgment in making any final determinations in product usage and technique. In doing so, the medical professional should rely on their own training and experience, and should conduct a thorough review of pertinent medical literature and the product's directions for use. Postoperative management is patient-specific and dependent on the treating professional's assessment. Individual results will vary and not all patients will experience the same postoperative activity level and/or outcomes.

View U.S. patent information at www.arthrex.com/corporate/virtual-patent-marking

arthrex.com

© 2022-12 Arthrex, Inc. All rights reserved. LB1-000418-en-US_A